

# HAIR WASHING TRAY

## NECESSARY EQUIPMENT

- A. HAIR WASHING TRAY
- B. Towels
- C. Shampoo
- D. Water supply or pitcher of warm water

## PROCEDURE

1. Position wheelchair (or chair) and client in front of sink with client's back to washbasin.
2. Place a towel around client's neck and shoulders.
3. Place tray strap over head with tray behind head, placing neck opening around neck (FIGURE 1).
4. Place client's forearm through strap and rest arm on abdomen to balance the tray on shoulders (FIGURE 2).
5. Make sure the end of the tray is far enough over the sink for the water to drain off the tray.
6. Wet hair and shampoo as usual.
7. Rinse hair well and let water flow down the tray into the sink.
8. Towel dry hair.
9. Remove the THE HAIR WASHING TRAY from client's shoulders.
10. Rinse, towel dry, and store THE HAIR WASHING TRAY.



MADE IN VIETNAM